RWSU EXPERT

The most versatile 4MHz
Radio Wave Surgical Platform

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PROXIMA – MEDICAL TECHNOLOGY

Technology is everywhere around us. Medical technology today is an integral part of the medicine and it meets us from the moment when we give blood sample for machine to count our blood cells, over non-invasive methods for observing the inside of our body by magnetic resonance imaging, to using high frequency electrical current, from electrosurgical generators, for cutting and coagulation of the tissue. Medical technology is of great importance for our society and our world.

Msc ee Saša Kocić, CEO

EXPERIENCE AND CONTINUANCE

For more than 20 years, we, PROXIMA Ltd., have been producing and distributing high quality medical technology. We are working to ensure that high quality medical technology is available and adequately implemented in Serbia and in the region.

MANUFACTURING AND QUALITY

As a manufacturer, we pay special attention to the details, to the quality and reliability of our technology. Production processes and quality control in accordance to ISO9001 and ISO13485 ensure longevity and reliability.

DESIGN AND DEVELOPMENT

Continuous development and improvements of our products and functionalities, thanks to our R&D department, make our products state of the art technology.

SERVICE AND TECHNICAL SUPPORT

Professional equipment, quick response and quality service procedure make our technical and service support the best in the class and represent additional value.

SALES AND DISTRIBUTION

Cooperation with world known manufacturers and distribution of the state-of-the-art medical technology makes our offer to the clients complete and makes possible the realization of even the most demanding projects.

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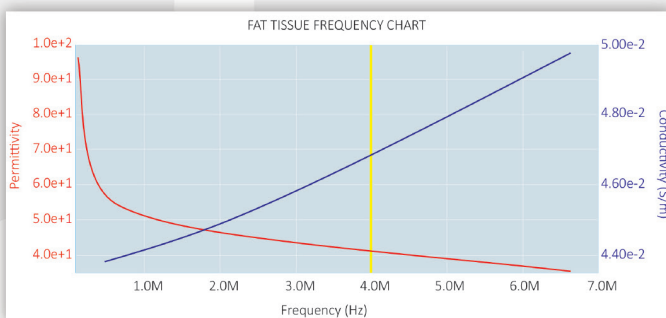
RADIOWAVE TECHNOLOGY

RADIOWAVE TECHNOLOGY

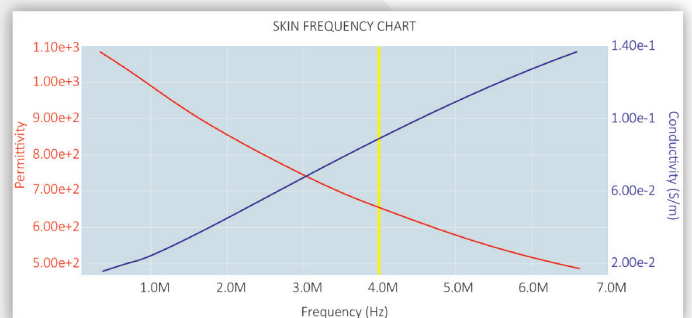
Electrical impedance of biological tissue decreases with increasing frequency of the electric field applied to it. This is mostly because the tissue cell membranes are poor conductors and are preventing electrical field from penetrating inside the tissue cells.

TISSUE PERMITTIVITY AND RESISTIVITY

At lower frequencies resistive component of cell membrane is dominant; at higher frequencies cell membrane gets capacitively coupled effectively short-circuiting cell membrane resistance, allowing electrical field to penetrate inside the cell structure. Therefore, effective tissue permittivity and resistivity is much lower at higher frequencies; this is exactly what happens at 4MHz frequency of RWSU Expert device.

EFFICIENCY

RWSU Expert device, when activated, delivers RF energy, induced by RF electrical field, to the tissue; at 4MHz instead of just passing through the intercellular space and cell membranes, energy passes through the whole cell structure too. As a result, RF energy is administered more gently and more evenly through the tissue structures creating desired effect (cut or coagulation) at the active electrode site, more efficiently than those effects on the tissue created by "conventional" electro-surgical devices.

Frekventna zavisnost kože
Skin frequency chart

INDICATION AREA OF RWSU EXPERT

- Gynecology
- Plastic surgery
- Maxillofacial surgery
- Dermatology (dermato-surgery)
- ENT
- Oral surgery

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BIPOLAR COAG – bipolarna koagulacija

BIPOLAR MICRO – bipolarna mikro koagulacija

SIX POWERFUL MODES OF OPERATION ENABLE PERFORMING EVEN MOST DEMANDING INTERVENTIONS:

SOFT CUT - clean cutting with minimal tissue damage

STRONG CUT - powerful cutting with stronger coagulation effect

SOFT COAG – soft contact coagulation

FORCED COAG – fulguration (forced coagulation)

BIPOLAR COAG – bipolar coagulation

BIPOLAR MICRO – bipolar micro coagulation

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SUPERIOR GRAPHIC USER INTERFACE:

- Complete interaction with device through the 7" color LCD touch sensitive display
- Interface with minimal menus and all parameters accessible immediately
- Choice of power activation by handswitch or double footswitch in monopolar mode and single footswitch in bipolar mode

SIGURNOST NA PRVOM MESTU:

- Implementirano sigurnosno kolo neutralne elektrode sa audio i svetlosnom signalizacijom i prekidom izlazne snage do uspostave regularnog kola neutralne elektrode

Korišćenje samolepljivih neutralnih elektroda

- smanjuje mogućnost kros-kontaminacije kod pacijenata

SAFETY FIRST:

- Implemented neutral electrode safety circuit with auditive and visual signalization and output power interruption until regular neutral electrode safety circuit is established

Usage of self-adhesive neutral electrodes

- lowers the possibility of cross-contamination between patients

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APPLIED ENGINEERING FOR USABILITY:

- Connections for active, bipolar and neutral electrode on the front, connections for footswitches and power supply on the back - enabling clean working space and no messy cables

- All accessories are with colored backlight during power activation or alarm of neutral electrode

- Design that makes RWSU Expert suitable for desktops as well as appropriate trolleys and easy moving between ORs

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ACCESSORIES

For RWSU Expert device a wide range of monopolar electrodes and bipolar instruments for multiple use are available:

Short (cca 7cm) monopolar electrodes (for dermatology, gynecology, plastic surgery, maxillofacial surgery, oral surgery, ENT...)

Long (cca 12cm) monopolar electrodes (for gynecology, plastic surgery, ENT...)

Bipolar forceps of different lengths and tip width
Bipolar clamps and scissors

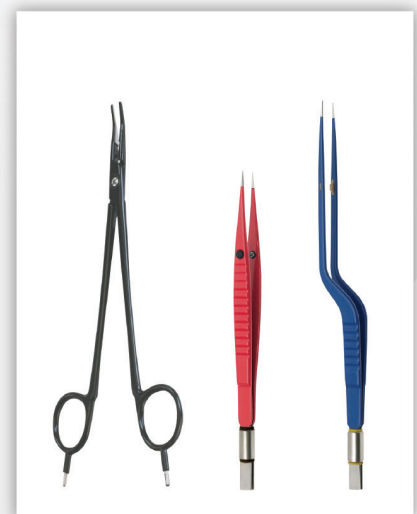
All electrodes and instruments are for multiple use and can be sterilized in autoclaves (hot steam sterilizers).

Širok spektar kratkih monopolarnih elektroda je dostupan
Wide range of short monopolar electrodes is available

Širok spektar dugačkih monopolarnih elektroda je dostupan
Wide range of long monopolar electrodes is available

Širok spektar bipolarnih instrumenata je dostupan
Wide range of bipolar instruments is available

TEHNIČKE KARAKTERISTIKE

TECHNICAL CHARACTERISTICS

NAPAJANJE POWER SUPPLY	230V 50Hz	
MODOVI RADA MODES OF OPERATION	Monopolar	Bipolar
	Soft Cut Strong Cut Soft Coag Forced Coag	Bipolar Coag Bipolar Micro
IZLAZNA SNAGA OUTPUT POWER	100W RMS @ 4MHz, sinusni oblik / sine wave	
RADNA FREKVENCIJA OPERATING FREQUENCY	4MHz	
SIGURONOSNO KOLO NEUTRALNE ELEKTRODE SAFETY CIRCUIT OF NEUTRAL ELECTRODE	Da / Yes	
PROGRAMSKA MEMORIJA PROGRAM MEMORY	Da, za 10 korisničkih programa sa nazivom programa Yes, for 10 user programs with program names	
NAPREDNA PODEŠENJA ADVANCED FEATURES	Kontinualni ili vremenski ograničen mod rada od 0.05sec do 5sec u koracima od 0.05sec za sve modove rada / Continuous or timed operation mode from 0.05sec to 5sec in 0.05sec steps for all modes of operation	

ŠIFRE ZA PORUČIVANJE

ORDER CODES

UREĐAJ RWSU EXPERT SA KOMPLETNIM STANDARDNIM SETOM AKSESORIJA RWSU EXPERT DEVICE WITH COMPLETE SET OF ACCESSORIES	PX-AP-2050
KOLICA SA TOČKOVIMA ZA RWSU EXPERT TROLLEY WITH WHEELS FOR RWSU EXPERT	PX-AP-2050-C1

PROXIMA - medical technology

office@proxima-medical.rs

+381 18 4 291 290, +381 18 4 291 288
Fax: +381 18 4 267 469

Moravska 7, Čokot
18250 Novo Selo, Niš, Republic of Serbia

1304

instagram.com/proxima_meditech

facebook.com/ProximaMedTech

Distributor: